

## SIDE RAIL BUMPER

In certain situations, it can be necessary to protect the restless and agitated clients from bumping into the side rails. These clients have a high risk of bumping into the side rails or pulling the VENDLET turning sheet away from the side rail, leaving hands and feet at risk of entrapment.

Side rail bumpers are easily mounted, and work up and down with the side rails. The side rail bumpers are mounted on the inside of the rails, so they don't affect the daily use of VENDLET.

The side rail bumpers are compatible with the following VENDLET models:

- VENDLET V5S
- VENDLET V5S+
- VENDLET V5S Speed Adjust
- VENDLET V5S+ Speed Adjust




Item No.: 1700009


The set consists of:

- 2 pcs PE foam sides
- 16 pcs Velcro

## Mounting instructions

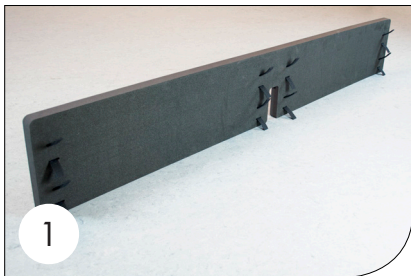

**1.** Place 8 pieces of Velcro in the cutouts in the foam side. The ends have to stick out on the side that has no VENDLET logo. **2.** Raise the bars and remove the turning sheet. **3.** Place the foam side on the inside of the bed rails. The side with the Vendlet logo has to face the mattress.

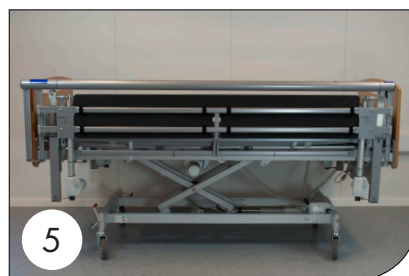

**4.** Let the foam side fall into place. **5.** Fix the Velcro around the support bars in the side rail. Repeat the process for the other side. Remount the turning sheet.

## Cleaning

The foam sides and the Velcro can be cleaned with warm water and regular detergents.

It is also possible to use alcohol, however in time it can discolor or bleach the foam sides.